| | |
|-----------------------------|----|
| Glossary | 3 |
| Description | 5 |
| Operating procedure | 6 |
| English | 7 |
| Français | 10 |
| Español | 13 |
| Deutsch | 16 |
| Italiano | 19 |
| Português | 22 |
| Čeština | 25 |
| Nederlandsk | 28 |
| Polskie | 31 |
| User assistance information | 34 |

GLOSSARY / GLOSSAIRE / GLOSARIO / GLOSSAR / GLOSSARIO / GLOSSÁRIO // GLOSÁŘ / WOORDENLIJST / GLOSARIUSZ

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-
-
-
-
-
-
-

PRESENTATION

DM ScanBox is available in one size only and is provided in a box containing a mesh pouch, a cushioning foam Reference 1001-001

INTENDED PURPOSE
DM ScanBox is designed to pair up with a smartphone in order to conduct scanning procedures performed with products marketed by Dental Monitoring.

INDICATIONS
DM ScanBox is intended to be used with DM Cheek Retractor during a scanning procedure guided by software developed and marketed by Dental Monitoring.

TARGET GROUPS
Target population: DM ScanBox is indicated for use for patients over the age of 6.

Intended users:
DM ScanBox is intended to be used by:

Healthcare professional in professional healthcare facilities, such as a dental practice.
Lay persons, patients themselves or third parties, in a non-healthcare environment, such as the patient's own home.
Type of use/reuse: DM ScanBox is reusable and can be used: Multiple times by a single patient.

CLINICAL BENEFITS

There is no direct clinical benefit for the use of DM ScanBox.

CLINICAL PERFORMANCES
DM ScanBox has no clinical performance as it has no direct action or effect on the patient.

TECHNICAL SPECIFICATIONS
The pairing of the DM ScanBox with the DM Cheek Retractor is carried out by magnetization.

SIDE EFFECTS
No known side effects.

CONTRA INDICATIONS
This device is not intended to be used on children under the age of 6.
The device should not be used on patients with pacemakers, defibrillators, or other medical devices incompatible with magnets (Br = 1.37T).

WARNINGS, PRECAUTIONS, NOTICES
Warning: No known warnings.
Precautions: Do not try to disassemble the device, you risk hurting yourself.
Do not clean in a dishwasher.
Do not clean in a washing machine.

en

Do not clean in an autoclave.
Do not clean the DM ScanBox with soap and water.
Do not put under running water.

The following population should be assisted by a third party in device use:
adults up to 12,
adults or children with visually impaired,
adults or children hearing impaired,
or any other condition that might prevent the patient from adequately operating the device.

DM ScanBox is provided in a non-sterile form. Refer to the cleaning procedure.
DM ScanBox is not sterilizable and does not submerge in any liquid.

Keep the magnets away from devices and objects that could be damaged by strong magnetic fields. If the door straps does not fit once the lid of the DM ScanBox covers your smartphone's screen, do not force, this could lead to smartphone damaging such as a broken screen. If you haven't already done it, take your phone case off and retry.

Do not try to disassemble the device, you risk hurting yourself.

Do not use the DM ScanBox if it is broken. Notices
This is not a toy. Children should be supervised to ensure that they do not play with the device.

INSTRUCTIONS FOR USE

PAIRING UP THE DM SCANBOX WITH YOUR SMARTPHONE
Hold the DM ScanBox with the front facing you. Place the lid ➊ by lightly pulling on the door strap ➋. Open your smartphone in the bottom left corner over the blue arrow ➌. The screen of your smartphone needs to be facing you. Latch your phone ➍. Turn on your phone case off if needed. Turn the box around in order to face the back of the DM ScanBox (make sure that the camera is inside the window). Push and hold the tuning button ➎ in order to align your phone with the lens alignment arrow ➏. Release the tuning button when you are finished.

PAIRING UP THE DM SCANBOX WITH YOUR DM CHEEK RETRACTOR
Put on your DM Cheek Retractor in accordance with its specific instruction For Use. Connect the DM ScanBox to the DM Cheek Retractor, you will hear a "click" when your DM ScanBox is properly attached to your retractor. In order to do so, align the magnetic plugs (see description part) with the metal inserts on the DM Cheek Retractor.

UNPAIRING THE DM SCANBOX
To remove the DM ScanBox, hold the retractor in place with your thumbs and pull the box forward without removing the DM Cheek Retractor from your mouth.

fr

INSTRUCTIONS FOR REUSABLE DEVICES

The following cleaning procedure must be followed for single-patient domestic use:
Clean the smartphone camera lens, if needed, following its manufacturer's recommendations.

Clean the DM ScanBox:
Before and after each use, clean the side of the DM ScanBox facing the DM Cheek Retractor (F in the picture below) by using a biocide wipe or a biocide spray paired with single-use paper towels.

Clean the rest of the DM ScanBox periodically by using a biocide wipe or a biocide spray paired with single-use paper towels.
Let the DM ScanBox air dry.
Note: Do not clean the DM ScanBox with soap and water. Do not put it under running water.

INSTRUCTIONS FOR DAMAGED PACKAGING

If the packaging is damaged or opened before use, return the pack to your healthcare facility.

STORAGE
The DM ScanBox should be away from solar radiation, compression and moisture. DM ScanBox can also be stored on the mesh pouch.

DISPOSAL
Dispose of it in accordance with your local recycling regulations. Do not throw into the nature.

LIST OF REQUIRED NON-PROVIDED PRODUCTS
DM Cheek Retractor (REF 1002-001, 1003-001, 1004-001). Dental Monitoring (REF 2001-001).

WARRANTY
One year from the date of original purchase.

TRAVEL
In the event of travel, please make sure the devices are not too tightly packed to prevent any deformation or breakage during transport.

en

INSTRUCTIONS FOR DAMAGED PACKAGING

If the packaging is damaged or opened before use, return the pack to your healthcare facility.

STORAGE
The DM ScanBox should be away from solar radiation, compression and moisture. DM ScanBox can also be stored on the mesh pouch.

DISPOSAL
Dispose of it in accordance with your local recycling regulations. Do not throw into the nature.

LIST OF REQUIRED NON-PROVIDED PRODUCTS
DM Cheek Retractor (REF 1002-001, 1003-001, 1004-001). Dental Monitoring (REF 2001-001).

WARRANTY
One year from the date of original purchase.

TRAVEL
In the event of travel, please make sure the devices are not too tightly packed to prevent any deformation or breakage during transport.

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