**DATE / DATA / TANGGAL / TARIKH / TARİH / 日期**

February, 2026 / Fevereiro de 2026 / Februari 2026 / Februari 2026 / Şubat, 2026 /  
2026 年 2 月

IFU\_LED\_A\_MD\_01\_OtherMarkets  
2026-02-16

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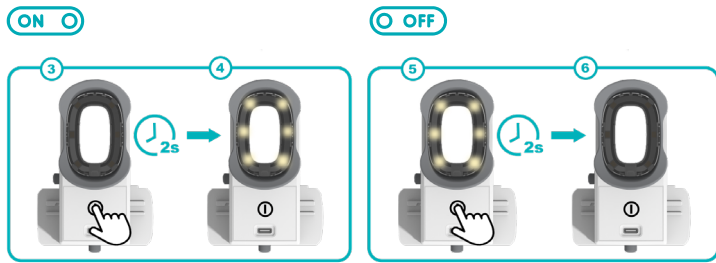
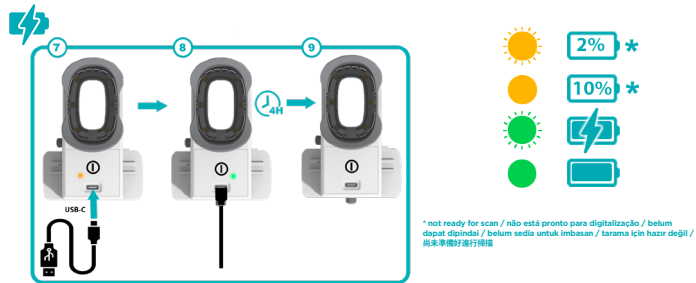
## PRESENTATION

The **LED Adapter** is available in a single-unit pack containing one **LED Adapter** (REF 1016-001) and one Instruction For Use.

It is designed exclusively for use with **ScanBox<sup>pro</sup>** devices also marketed by Dental Monitoring.

The **LED Adapter** is battery-powered, enabling LEDs to be switched on and off by touching for 2 seconds a touch-sensitive button. The battery of the **LED Adapter** can be reloaded using a Type-C charger which is not provided in the **LED Adapter** pack.

## INTENDED PURPOSE

The **LED Adapter** is intended to be used as an accessory of the **ScanBox<sup>pro</sup>** to provide illumination of the mouth to take pictures of the intraoral cavity using a smartphone.

The **LED Adapter** is used when smartphones are not directly compatible with the **ScanBox<sup>pro</sup>**.

## INDICATIONS

The **LED Adapter** is intended to be used as part of a scanning procedure using **ScanBox<sup>pro</sup>** and guided by software, developed and marketed by DentalMonitoring, to monitor intraoral health and orthodontic treatments.

## TARGET GROUPS

Target population: **LED Adapter** is indicated for patients over the age of 6.

Intended users: **LED Adapter** is intended to be used by:

- Healthcare professional in professional healthcare facilities, such as a dental practice.
- Lay persons over the age of 6 in a non-healthcare environment, such as the patient's own home.

Type of use/reuse: **LED Adapter** is reusable and can be used by multiple patients, multiple use.

### III. INSTRUCTIONS FOR USE

#### CLINICAL BENEFITS

There is no direct clinical benefit for the use of the **LED Adapter**. The **LED Adapter** is intended to be used to ease the taking of scans using **ScanBox<sup>pro</sup>** and that will be processed by DentalMonitoring.

#### CLINICAL PERFORMANCES

The **LED Adapter** has no clinical performance as it has no direct action and effect on the patient.

#### TECHNICAL SPECIFICATIONS

|                       |                          |
|-----------------------|--------------------------|
| <b>POWER SUPPLY:</b>  | Lithium-ion Battery 3,7V |
| <b>RECHARGE:</b>      | USB Type C - 5V DC 1A    |
| <b>IP PROTECTION:</b> | IP34                     |

|                                       |                                                        |                                                       |
|---------------------------------------|--------------------------------------------------------|-------------------------------------------------------|
| <b>STORAGE CONDITIONS:</b>            | (-20)°C to 60°C (-4°F à +140°F)                        |                                                       |
| <b>CHARGING/OPERATING CONDITIONS:</b> | 0°C to 35°C (32°F à 95°F)                              |                                                       |
| <b>STANDARDS APPLIED:</b>             | ISO 14971<br>ISO 13485<br>IEC 60601-1<br>IEC 60601-1-2 | IEC 60601-1-6<br>IEC 60529<br>IEC 62133-1-2<br>UN38.3 |

This device has been tested for EMC according to IEC 60601-1-2:2014 + A1:2020 and meets the emission and immunity requirements for use in residential environments. The detailed intended use environment and test specifications are available by scanning the QR code below.

### III. INSTRUCTIONS FOR USE

#### SIDE EFFECTS

No known side effects.

#### CONTRAINDICATIONS

**LED Adapter** should not be used in the event of injured skin or injured mucous membrane.

**LED Adapter** is not intended for surgical or post-surgery use, nor to take intra-oral pictures with products other than those marketed by DentalMonitoring.

#### WARNINGS, PRECAUTIONS, NOTICES

##### • Warnings

**LED Adapter** should not be put directly into the patient's mouth.

Do not stare directly at the **LED Adapter** light.

**LED Adapter** should not be cleaned using a dishwasher or water.

**LED Adapter** should not be cleaned under water nor under hot temperature water.

**LED Adapter** is not intended to be used in an oxygen-rich environment.

**LED Adapter** should not be exposed to extreme temperature/humidity. Respect temperature/humidity limits:

- Storage Temperature limit from -20°C to 60°C (-4°F à +140°F)
- Operating / Charging Temperature limit from 0°C to 35°C (32°F to 95°F)
- Humidity limitation from 35% to 70%
- 700 hPa to 1060 hPa (altitude 0 to 3,000 m / 0 to 9,843 ft)

No modification of the **LED Adapter** is allowed.

Do not try to replace the Lithium-ion battery of the **LED Adapter**.

Do not disassemble the **LED Adapter**.

Do not operate the device while charging.

### III. INSTRUCTIONS FOR USE

Use only a compatible charger rated at 5V DC / rated up to 3A with a USB-C connection.

During normal operation, certain surfaces of the equipment (e.g. areas near the LED module) may reach a maximum temperature of 42.5 °C (107.5°F).

Charging may warm the **LED Adapter**. If it becomes unusually hot, unplug immediately and stop use.

Unplug the **LED Adapter** once fully charged. Continuous charging may cause battery damage or fire.

If leakage from the Lithium-ion battery is detected on the **LED Adapter**, do not use the **LED Adapter** anymore. The electrolyte contained in the battery is corrosive and may cause chemical burns or irritation.

Replacement of the battery by inadequately trained personnel could result in short-circuit, battery overheat leading to fire.

If the **LED Adapter** is damaged and/or in case of signs of wear, such as visible cracks or fractures, and notice discoloration or warping the device should not be used.

In order to regulate the requirements for ElectroMagnetic Compatibility with the aim of preventing unsafe product situations, the IEC 60601-1-2:2014 + A1:2020 standard has been implemented:

- Use of this equipment adjacent to or stacked with other electronic devices that are not provided or recommended by Dental Monitoring, should be avoided because it may result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than provided or recommended by Dental Monitoring could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

### III. INSTRUCTIONS FOR USE

- Keep portable radiofrequency communication equipment, such as other wireless devices not recommended by Dental Monitoring, at least 30 cm (12 inches) away from the power supply or battery charger (including cables). Getting too close may affect how the device works.

The use of compatible smartphones does not affect the device's operation.

#### • Precautions

**LED Adapter** should not be used without a **ScanBox<sup>pro</sup>**. Be sure to read the instruction for use of the **ScanBox<sup>pro</sup>** before using the **LED Adapter**.

The following population should be assisted by a third party in device use:

- children up to 12;
- visually impaired adults or children;
- hearing impaired adults or children;
- adults or children with any other condition that might prevent the patient from adequately operating the device.

If the product has been stored in cold or hot conditions, allow it to return to room temperature before use.

The **LED Adapter** contains tetrabromobisphenol A (CAS n°79-94-7) in quantity superior to 0.1% w/w.

#### • Notices

This is not a toy. Children should be supervised to ensure that they do not play with the device.

### INSTRUCTIONS BEFORE USE

Allow the device to charge fully before first use.  
Clean the device according to the cleaning instructions before first use.  
Follow the instructions directly displayed on the DM App to ensure your smartphone is compatible with **LED Adapter**.

#### DIRECTIONS FOR USE

##### Description

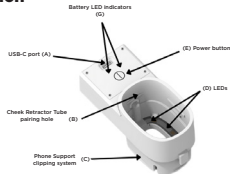
Figure 1: LED Adapter

##### Pairing the LED Adapter with your ScanBox<sup>pro</sup>

The **LED Adapter** may be required depending on your phone model; the application automatically detects whether the **LED Adapter** is required and informs you accordingly.

Connect the **LED Adapter** to the Phone support by inserting the end of the **LED Adapter** with the clipping system (C) into the tube pairing hole on the **phone support** ①. You will hear a "click" when the **LED Adapter** is properly attached.

Then connect the **Cheek Retractor Tube** to the **LED Adapter** by inserting the end of the tube with the clipping system into the **LED Adapter** pairing hole ②. You will hear a "click" when the **Cheek Retractor Tube** is properly attached.

Then connect the **Phone Support** to the phone. Refer to the IFU for the **ScanBox<sup>pro</sup>**.

##### Switching on the LED Adapter

Once the phone is connected to the **Phone support**, the **LED Adapter**

and the **Cheek Retractor Tube**, touch the power button for 2 seconds to switch on the LED ③.

##### Switching off the LED Adapter

Once you have finished your scan, touch the power button (E) for 2 seconds to switch off the LED ⑤.

##### Unpairing the LED Adapter and the ScanBox<sup>pro</sup>

First remove the **Phone Support** from the phone. Refer to the IFU for the **ScanBox<sup>pro</sup>**.

Then remove the **Cheek Retractor Tube** from the **LED Adapter**, holding the **Cheek Retractor Tube** in one hand and the rest of the system in the other and pulling the **Cheek Retractor Tube** forward.

Then remove the **LED Adapter** from the **Phone Support**, holding the **Phone Support** in one hand and the **LED Adapter** in the other and pulling the **LED Adapter** forward.

The **LED Adapter** and the **Phone Support** can be stored always connected.

##### Recharging the LED Adapter

Use only an USB-C charging cable less than 3 metres (9.84 feet long) and a 5V DC charger.

Connect the charging cable to the device's charging port (A) on the front of the **LED Adapter** ⑦. Ensure there is no debris or moisture in the port before connecting.

Plug the charger into a standard electrical outlet (100–240 V AC). Make sure the device is placed on a flat, dry surface during charging. Avoid charging on soft materials like bedding or towels to prevent overheating.

The charging indicator will flash to confirm charging ⑧. Refer to the indicator guide to check the charging status.

Unplug the charger once the device is fully charged (approximately 3h40). The light will turn green and no longer flashes ⑨.

Avoid overcharging when possible to prolong battery life.

### III. INSTRUCTIONS FOR USE

#### INSTRUCTIONS FOR REUSABLE DEVICES

The following cleaning procedure must be followed:

- Before and after each use, clean the **LED Adapter** by using a biocide wipe or a biocide spray paired with single-use paper towels.
- Let the **LED Adapter** air dry.

#### INSTRUCTIONS FOR DAMAGED PACKAGING / DAMAGED PRODUCT

If the packaging is damaged or opened before use, return the pack to your healthcare facility or to DentalMonitoring by contacting: [support@dental-monitoring.com](mailto:support@dental-monitoring.com).

#### INSTRUCTIONS FOR VERIFICATION

The device must be properly assembled before use with **ScanBox<sup>pro</sup>**.

The LED indicator color should not be in orange before use. If the LED Indicator is orange, recharge the **LED Adapter**.

#### STORAGE

- Store in a cool, dry place.
- Keep away from direct sunlight, heat sources, or open flames.
- Do not expose the device to temperatures above 60°C (140°F).
- Avoid high humidity environments (e.g., bathrooms without ventilation).
- Ensure the product is kept in the **ScanBox<sup>pro</sup>** to protect from moisture and dust.
- Do not store near water or in areas at risk of condensation or splashing.

### III. INSTRUCTIONS FOR USE

#### DISPOSAL

The **LED Adapter** must not be disposed of with household waste. Follow local regulations for lithium battery disposal.

##### Removal of the battery only by qualified personnel:

1. Unscrew the 4 screws.
2. Remove the **LED Adapter** lid.
3. Unscrews the 2 PCB's screws.
4. Remove the PCB & disconnect the battery.
5. Unstick the battery.

#### LIFETIME

The lifetime of the LED Adapter is 5 years.

#### TRAVEL

In the event of travel, the device should be kept in the mesh pouch provided with the **ScanBox<sup>pro</sup>**.

Please make sure the devices are not too tightly packed to prevent any deformation or breakage during transport.

#### LIST OF REQUIRED NON-PROVIDED PRODUCTS

**DentalMonitoring** (REF 2001-001)

**ScanBox<sup>pro</sup>** (REF 1009-001, REF 1010-001, REF 1011-001, REF 1012-001,

### III. INSTRUCTIONS FOR USE

REF 1014-001, REF 1015-001)

Type-C charger and an USB-C charging cable

#### USER ASSISTANCE INFORMATION

In case of any unwanted events or issues in relation to the device please contact [support@dental-monitoring.com](mailto:support@dental-monitoring.com)

Please report any serious incident that occurred in relation to the

device to [support@dental-monitoring.com](mailto:support@dental-monitoring.com) and to the local competent authority for medical devices.

To find your local user assistance phone number:  
[www.dental-monitoring.com](http://www.dental-monitoring.com)

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